


# D Free<sup>®</sup>

## Personal

## Toilet timing made easy.

The first wearable device for urinary incontinence – notifies you when it's time to go to the bathroom.

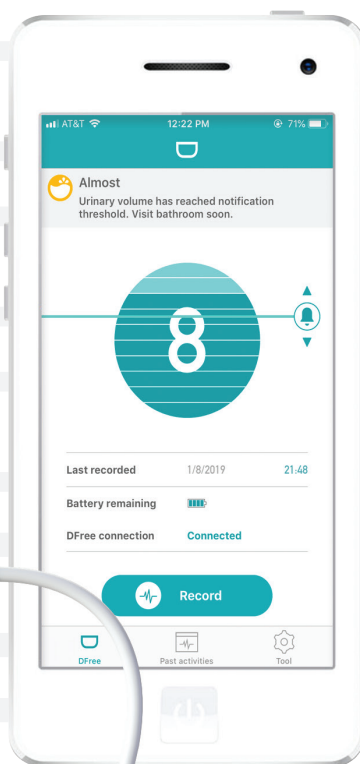


Used in over 500 senior care facilities since 2017

# Do you have any of these challenges?

- Can't hold urine to make it to the bathroom
- Have difficulty controlling the urge to urinate
- Worried about overly active bladder and accidents
- Afraid of going outside due to a lack of bladder control
- Worried about bladder control but don't want to rely on pads yet
- Can't feel the urge to urinate


**D Free**<sup>®</sup>  
Personal

## What does DFree do?

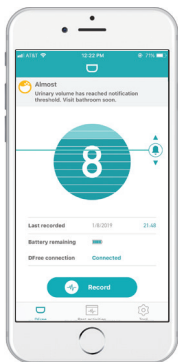

### NOTIFIES WHEN YOU NEED TO GO

DFree predicts urination timing and sends notifications to your phone or tablet.

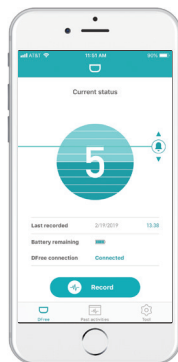

### SHOWS HOW FULL YOUR BLADDER IS

Whether you are at home or outside, monitor your bladder in real time.




### KEEPS TRACK OF YOUR BATHROOM VISITS

Know the average time in between bathroom visits and how often you need to go.

**D Free**<sup>®</sup>  
Personal

## How does DFree work?

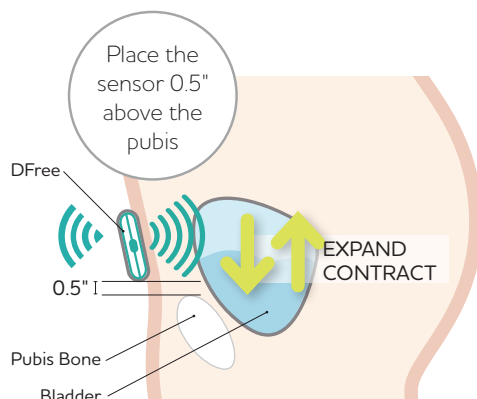

### SENSES BLADDER WITH ULTRASOUND

DFree detects the changes in bladder size and predicts urination timing. DFree uses harmless and non-invasive ultrasound sensor.



### CONNECTS TO SMART DEVICES

DFree sends alerts to your smart phone or tablet when it is time to go to the bathroom. Set notification threshold so you have ample time to go.

# What Our Customers Say...



“ I can't wait to go out again!

**BEFORE:** When I go out, I was afraid of having accidents and not knowing when to go to the bathroom. I would refrain from drinking too much water but I was still concerned.

Woman, 70's

**AFTER:** After analyzing data from DFree®, I realized that I can hold urine for at least 2-3 hours after drinking water. Now I can go out without worrying about finding a bathroom.



“ Essential toilet training for my child with special needs.

**BEFORE:** Our child with special needs wears a diaper because he can't express when he needs to go.

Man, 40's

**AFTER:** Using DFree, we are able to take our son to the bathroom when his bladder is full. Our goal is to train him to use the bathroom and eliminate the need to use diapers.



“ I no longer have to worry about finding a bathroom.

**BEFORE:** After an accident, I had to use a catheter due to sensory loss. When I go out, I am always worried about not knowing when to go to the bathroom.

Woman, 30's

**AFTER:** With DFree telling me when my bladder is full, I am no longer worried about finding a bathroom when I am outside.



“ I don't have to worry about my father falling at night.

**BEFORE:** I am taking care of my father who wakes up often during the night and falls on his way to the bathroom.

Man, 50's


**AFTER:** Using DFree, I know when my father needs to go to the bathroom so I can assist him. I am no longer worried about my father falling and hurting himself during the night.



# Product Information




DFree  
(Dfree-U1P)

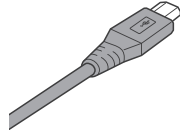

USB Cable  
(for charging)



Ultrasound Gel\*  
(2 oz.)

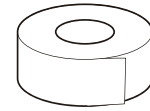

Medical Tape\*  
(1 roll)

\* Ultrasound gel and medical tape sold separately.

## Device Specifications

	Size (inches)	Weight (oz)	Power supply	Battery life
Main unit	3.27"(W) x 3.15"(D) x 1.3"(H)	2.56	Lithium-ion rechargeable battery	Approx. 24 hours of use (Fully charged in 4 hours)
Sensor	2.13" (W) x 1.38" (D) x 0.48" (H)	0.64		

## Application Specifications

DFree app currently supports the following devices:

[iPhone] iPhone 5 or later

[iPod] iPod Touch 6th Gen or later

[iPad] iPad 5, iPad Air, iPad Mini2, iPad Pro or later

[Android] phones, tablets

\*The DFree app may be downloaded for free from the App Store and Google Play. The app supports iOS devices (iPhone, iPad, iPod) running iOS version 10 or later and Android devices (phones, tablets) running Android 4.4 or later.

## Tracking Bladder with DFree Sensor

Ultrasound sensor on DFree may not be able to accurately track and collect data from the bladder with certain types of users. These users may have the following conditions:


- Excessive fat in the lower abdomen area
- Loose or flabby skin in the lower abdomen area
- Scar(s) from previous surgery or injury
- Problems accumulating urine in the bladder
- Prone to rashes from use of medical tape and ultrasound gel
- Pregnant
- Underage (6 or younger - bladder may be too small to accurately collect data)

## OUR MISSION

Empowering people with urinary incontinence to live without worries.

DFree can help millions of seniors, disabled people and others who need to manage incontinence. We believe that having control of your senses is the key to independence and living your life to the fullest.

## AWARDS

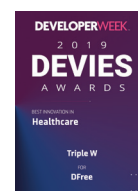

Triple W - DFree | 5795 Kearny Villa Road, San Diego, California 92123

DFree is not a medical device and is not intended to be used in the diagnosis, prevention, or treatment of diseases or medical conditions.



Contact Customer Support

[www.dfreeus.biz](http://www.dfreeus.biz)

833-DFREE US (833-337-3387)

[sp-us@www-biz.co](mailto:sp-us@www-biz.co)

Monday - Friday 10:00am-5:00pm PST (excluding holidays)